

M O K V O K  
E Y E P A T C H

---

S I Z E  
G U I D E

---

2 0 2 3

---

## **Instruction Guide for Printing and Trying On a Paper Eye Patch Template**

Note: This guide provides step-by-step instructions with explanations for printing a paper eye patch template, trying it on, and ensuring the best fit.

Materials Needed:

- Printer
- Ruler
- Scissors
- Paper (sturdy enough to hold its shape)

### **Step 1: Print the Template**

Print next page in its size on Page A4. To ensure that the page is printed in its original size, place your credit card or ID card in the box below. If it exactly coincides with its outline, you can continue.

### **Step 2: Cut Out the Template**

Using scissors, carefully cut along the outer edge of the printed eye patch template. The resulting shape should be the paper eye patch template.

### **Step 3: Try On the Paper Template**

Hold the paper template up to your eye area without pressing it against your skin. Position the template so that it covers the eye area properly and comfortably.

### **Step 4: Assess Fit and Comfort**

Look in a mirror and assess how the paper template fits around your eye. Adjust the template's position to find the most comfortable and suitable fit. Check the fit and comfort, making sure it covers the eye area adequately.

### **Step 5: Using the Template for a ordering Eye Patch**


If you're satisfied with the fit of the paper template, you can use it to order the right size on our website. Please remember that this paper template serves as a visual guide to help you find the right size level for an eye patch.

---



---

**Print me!**

Right

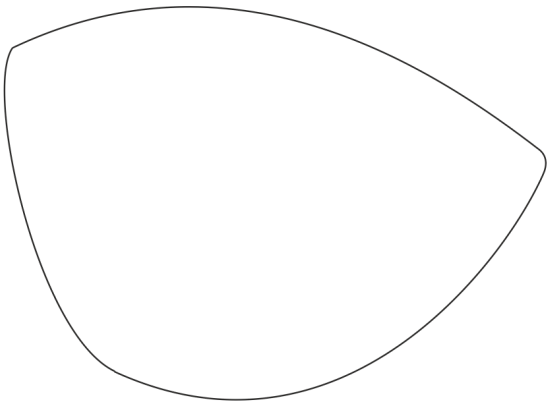

Left

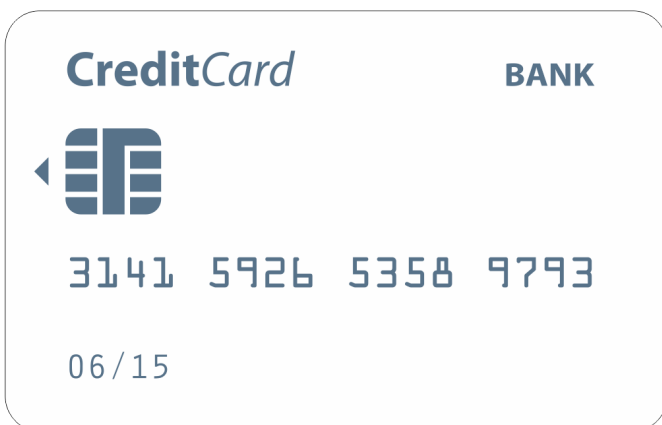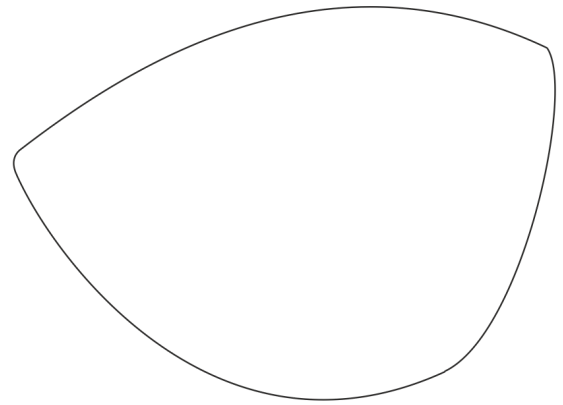


Place your credit card or ID card here to see if the page has been printed in its original size.

---

---

## Patch Types

Remember a convex eye patch extends outward away from the eye.  
A Flat patch will fit under glasses.

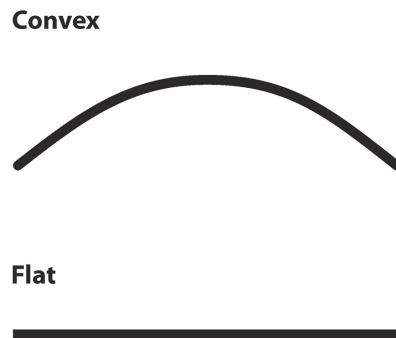

**Convex Leather Medical Eye Patch:** It is designed to protrude slightly from the eye area. The curvature allows for more space between the patch and the eye itself. This type of patch might be used to prevent direct contact between the patch and the eye, providing more comfort for the wearer.


**Flat Leather Medical Eye Patch:** This type of eye patch has a flat surface and is usually designed to cover the eye evenly. It sits flush against the skin and provides a consistent coverage over the eye area.

---

---

## How to measure your eye for an Eye Patch!

Measure from the top of the eyebrow down to the upper part of the cheekbone. Then measure from the outer edge of the eye to the middle bridge of the nose. This will give you an approximate measurement to choose from. Always double check your measurements. Different people have different shapes.

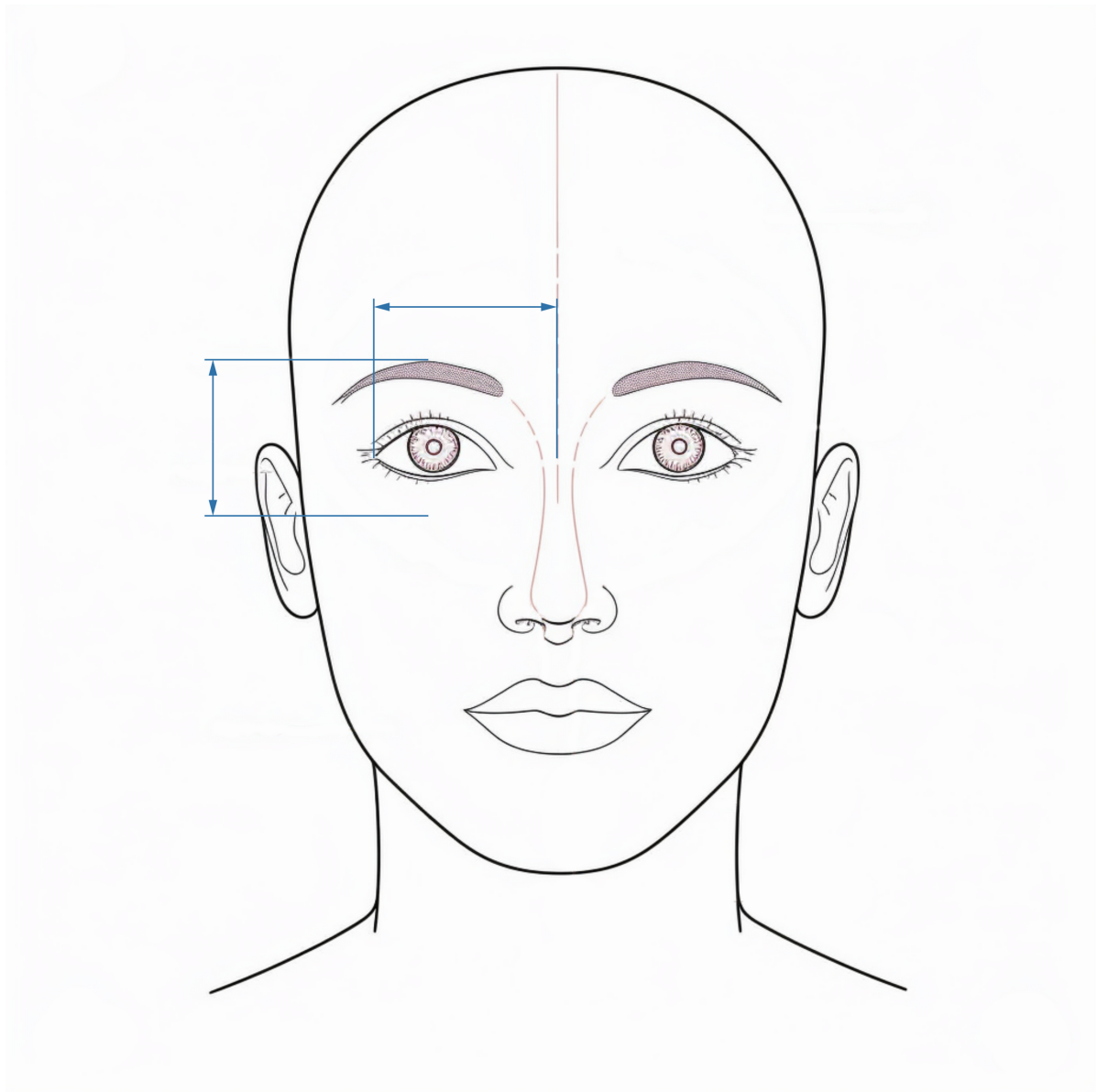

---

## Elastic band length:

An elastic strap with adjustable features is designed to accommodate all head sizes. This versatile design ensures a comfortable and secure fit for everyone.

The length of the strap's circumference can be adjusted from 50 to 66 centimeters (19.5 to 26 inches).

The elastic strap is equipped with adjustable mechanisms that allow you to customize the tightness of the strap according to your specific head circumference. This means that whether you have a smaller head or a larger one, you can easily adjust the strap to achieve the perfect fit.

The benefit of an elastic strap with adjustable features is that it provides a snug yet gentle hold on the head, preventing the eye patch from slipping or causing discomfort. This feature makes the eye patch suitable for a wide range of individuals, ensuring that the patch stays in place without being too tight or too loose.

